

RADIESSE®



Don't
forget your
hands



Collagen



Elastin



Contour



Lift



The **best filler for hand rejuvenation**
according to NewBeauty®

Please see **Important Safety Information**
throughout and **Instructions for Use** at
[Radiesse.com](https://www.radiesse.com)



Actual patient. Individual results may vary.

RADIESSE®

IMPORTANT CONSUMER RADIESSE® and RADIESSE® (+) SAFETY INFORMATION (continued)

Injection of RADIESSE® (+) into the jawline may temporarily alter jaw function.

The microspheres in RADIESSE® and RADIESSE® (+) can be seen in X-rays and CT Scans. It is very important that you tell your health care provider that you have had RADIESSE® or RADIESSE® (+) dermal filler.

If you have a history of herpes, you may experience a herpes breakout after receiving RADIESSE® or RADIESSE® (+).

Injection in the back of the hand may result in temporary difficulty performing activities. RADIESSE® may cause nodules, bumps or lumps in the back of the hand and can last up to 1 year.

You should minimize strenuous activity and avoid extensive sun or heat exposure for about 24 hours after treatment and until any swelling or redness has resolved.

What should I tell my doctor before using RADIESSE® or RADIESSE® (+)?

Tell your health care provider if you are taking blood thinners or medicines that can interfere with the clotting of blood, such as aspirin or warfarin. These medicines might make it more likely that you will experience bruising or bleeding at the injection site. Tell your health care provider if you have any diseases, injuries or disabilities of the hand, if you have a history forming large, raised scars or if you have had any other skin treatments such as skin peels.



Hands and ageing

Next to the face, the hands are one of the most visible areas of the body and are subjected to external ageing factors such as sun exposure, weather and friction forces.¹¹ These external factors are coupled with internal factors such as nutrition, overall health and medication to affect the overall aesthetic appearance of the hands.¹²

Key features of hand ageing^{11,12}

Ageing of the hand is characterized by:

Changes in skin texture which includes:

- + Irregular surface pigmentation
- + Thinning of the skin
- + Decrease in skin elasticity

Volume loss leading to:

- + Increased exposure of tendons
- + Pronounced veins
- + Bony appearance

What are RADIESSE® and RADIESSE® (+)?

RADIESSE® and RADIESSE® (+) are dermal fillers that are used for smoothing moderate to severe facial wrinkles and folds, such as nasolabial folds (the creases that extend from the corner of your nose to the corner of your mouth). RADIESSE® is also used for correcting volume loss in the back of the hands. RADIESSE® (+) is also used for improving moderate to severe loss of jawline contour in adults over 21 years old.

IMPORTANT CONSUMER RADIESSE® and RADIESSE® (+) SAFETY INFORMATION

Who should not use RADIESSE® or RADIESSE® (+)?

You should not use RADIESSE® or RADIESSE® (+) if you have an allergy to any component of the product, if you have a history of severe allergies, if you have a bleeding disorder, or if you are pregnant or breastfeeding. You should not use RADIESSE® (+) if you have an allergy to lidocaine or medicines like it and 1 year in the hand.

What is the most important information I should know about RADIESSE® and RADIESSE® (+)?

One of the risks with using these products is unintentional injection into a blood vessel. The chances of this happening are very small, but if it does happen, the complications can be serious, and may be permanent. These complications, which have been reported for facial injections, can include vision abnormalities, blindness, stroke, temporary scabs, or permanent scarring of the skin.

Measuring

Measuring the aging process¹⁰



No loss of fatty tissue



Mild loss of fatty tissue
slight visibility of veins



Moderate loss of fatty tissue;
mild visibility of veins and tendons



Severe loss of fatty tissue;
moderate visibility of veins and tendons



Very severe loss of fatty tissue;
marked visibility of veins and tendons

IMPORTANT CONSUMER RADIESSE® and RADIESSE® (+) SAFETY INFORMATION (continued)

If you have changes in your vision, signs of a stroke (including sudden difficulty speaking, numbness or weakness in your face, arms, or legs, difficulty walking, face drooping, severe headache, dizziness, or confusion), white appearance of the skin, or unusual pain during or shortly after treatment, you should notify your health care practitioner immediately.

RADIESSE® provides long-lasting improvement for volume loss in the hands

- + Restores a more youthful appearance⁹
- + Reduces the appearance of tendons and veins⁹
- + Delivers smooth, natural-looking results that last up to a year in most patients⁹

BEFORE



AFTER



*Actual patient. 2 weeks after being injected with 3.0 cc of Radiesse in the hands. Individual results may vary.

Confidence to come back

XPERIENCE TM
BEAUTY WITH BENEFITS

Get 50 Xperience+ bonus points when you return to your preferred participating provider



*Bonus points cannot be earned on the first visit
“Beauty with Benefits” is related to the program and not intended to imply efficacy.

IMPORTANT CONSUMER RADIESSE® and RADIESSE® (+) SAFETY INFORMATION (continued)

As with all procedures that involve an injection through the skin, there is a risk of infection.

Do not use RADIESSE® or RADIESSE® (+) if you have a skin infection until it has healed.

It is not known if RADIESSE® or RADIESSE® (+) is safe or effective in the lips, or in the area around the eyes.

It is not known if RADIESSE® or RADIESSE® (+) is safe or effective beyond 3 years in the face and 1 year in the hand.

RADIESSE®

Ask your provider about the Radiesse® approach

- Unique CaHA injectable treatment for the immediate improvement you want, plus the collagen and elastin production you need¹⁻⁶
- Clinically proven safe and predictable results³
- Long-lasting results—a year or more in many patients^{7,8}

IMPORTANT CONSUMER RADIESSE® and RADIESSE® (+) SAFETY INFORMATION (continued)

What are the most common adverse events with RADIESSE® or RADIESSE® (+)?

The most common adverse events seen in clinical studies of RADIESSE® used in the hands include bruising, redness, swelling, pain, itching, nodules or bumps/lumps, difficulty performing activities, loss of sensation and other local side effects. The most common adverse events seen in clinical studies of RADIESSE® or RADIESSE® (+) used in the face include bruising, redness, swelling, pain, itching, difficulty chewing, and other local side effects.

These are not all of the possible side effects with RADIESSE® or RADIESSE® (+). Merz collects information about adverse events seen with RADIESSE® and RADIESSE® (+) outside of clinical studies. These events are included in the RADIESSE® and RADIESSE® (+) Patient Information Guide based on an assessment of seriousness and potential causal relationship to RADIESSE® or RADIESSE® (+). Please see the Patient Information Guide available at www.radiesse.com for list of these events. Tell your health care provider about any side effects that bother you or do not go away.

Important: For full safety information, please visit www.Radiesse.com or call MyMerz Solutions at 1-844-469-6379

RADIESSE® and RADIESSE® (+) are available by prescription only.

References:

1. Radiesse (+) Instructions for use]. Franksville, Wt. Merz North America, Inc; 2021.
2. Radiesse Instructions for use]. Franksville, WI. Merz North America, Inc; 2016.
3. Loghem JV, Yutskovskaya YA, Philip Werschler W. Calcium hydroxylapatite: over a decade of clinical experience. *J Clin Aesthet Dermatol*. 2015;8(1):38-49.
4. Tzikas TL. A 52-month summary of results using calcium hydroxylapatite for facial soft tissue augmentation. *Dermatol Surg*. 2008;34 Suppl 1:S9-15.
5. Yutskovskaya Y, Kogan E, Leshunov E. A randomized, split-face, histomorphologic study comparing a volumetric calcium hydroxylapatite and a hyaluronic acid-based dermal filler. *J Drugs Dermatol*. 2014;13(9):1047-1052.
6. González N, Goldberg DJ. Evaluating the effects of injected calcium hydroxylapatite on changes in human skin elastin and proteoglycan formation. *Dermatol Surg*. 2019;45(4):547-551.
7. Moers-Carpi M, Vogt S, Santos BM, Planas J, Vallve SR, Howell DJ. A multicenter, randomized trial comparing calcium hydroxylapatite to two hyaluronic acids for treatment of nasolabial folds. *Dermatol Surg*. 2007;33 Suppl 2:S144-151.
8. Bass LS, Smith S, Busso M, McClaren M. Calcium hydroxylapatite (Radiesse) for treatment of nasolabial folds: long-term safety and efficacy results. *Aesthet Surg J*. 2010;30(2):235-238.
9. Radiesse Hands Instructions for use]. Franksville, WI. Merz North America, Inc; 2019.
10. Carruthers, A et. al. A Validated Grading Scale Hand Grading Scale.
11. Sattler et al. *Akt Dermatol* 2014,40: 445-451.
12. Berry C. 2014 <http://aestheticsjournal.com/fleature/hand-rejuvenation-1>.

MERZ AESTHETICS®

©2023 Merz North America Inc. All rights reserved. MERZ AESTHETICS and the XPERIENCE (+) BEAUTY WITH BENEFITS logo are registered trademarks of Merz Pharma GmbH & Co. KGaA in the U.S. RADIESSE is a registered trademark of Merz North America, Inc. in the U.S. 2023 NewBeauty Award, Radiesse, The Best Filler for Hand Rejuvenation. US-RAD-2300039